


SofPulse, WoundPulse, and OrthoHalo Operating Instructions

Your doctor will place and activate your Endonovo targeted Pulsed ElectroMagnetic Field (tPEMF) therapy device after surgery. During automatic treatments it is important to keep your device in place except when showering and bathing.

Operating Instructions

Automatic Treatment

- The tPEMF device will deliver a 15-minute treatment every two-hours.
- During active treatment two therapy indicator lights will blink every second.
- In between treatments one therapy indicator light will blink every five seconds.

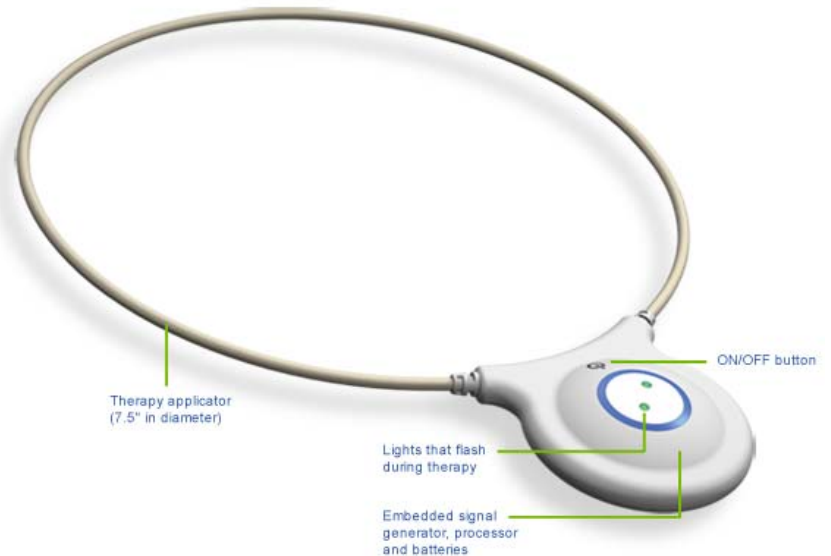

Turning Device Off

- To turn the device off, press the power button. The device is off when the indicator lights do not flash.
- When the device is turned on again, it will begin a new 15-minute active treatment.

Ending Treatment

- Treatment is complete when the indicator lights blink rapidly (about 8.5 days) or your surgeon advises you to discontinue treatment.

Applicator Etiquette

Bending or elongating the applicator too much will hinder effectiveness. Keeping the applicator as circular as possible provides the best field of treatment.

